


EN

Instructions for use


REF 40030

REF 40020

MD C E

 -10°C
min. 30°C
max.

What is the RheoDol® Oral Hygiene

RheoDol® Oral Hygiene offers patient groups, particularly those in intensive care and oncology, an innovative product component system based on biomolecular science for oral cleansing ^(A), germ reduction ^(B), and wound care ^(C). For these patient groups, as well as others, starting from childhood, RheoDol® Oral Hygiene represents a permanently usable, cost-effective, and optimal product combination.

The medical devices / care products can be used in the Oral Hygiene ABC Oral Cleaning ^(A), Germ Reduction ^(B) and Wound Care ^(C) as well as in individual applications.

Oral Hygiene ABC is applied according to a 3-step system. Further information on the following pages.

single patient use**3-step application of the RheoDol[®] Oral Hygiene**

1. Teeth
2. Tongue
3. Palate
4. Gums
5. Oral mucosa
6. Lips
7. Dental Implants
8. Dentures

A**RheoDol[®] Oral Hygiene Gel plus****Oral cleaning**

Use pea-sized portion of gel in the morning, evening and after meals to brush the teeth and clean the entire oral cavity and tongue. For tongue cleaning and the removal of pathological coatings simply use cotton or oral care sticks. If possible, do not rinse; spread remaining gel in the oral cavity and on the tongue. Natural dentition, implants and removable dentures can also be cleaned with the gel.

Application

- Remove lid from the bottle.
- Remove safety nipple.
- Apply pea-sized portion of gel by squeezing the bottle head on swap.
- Apply the gel to the oral cavity.
- Write the date of opening on the bottle.
- Store dry, away from light at room temperature (12 months after opening)

B**RheoDol® Oral Hygiene Liquid plus****Oral germ reduction**

Apply rinsing solution (Liquid) after oral cleaning or more often for inflammatory processes. Dose with measuring cup approx. 5-10 ml to rinse or swab (e.g. with cotton stick or oral swab) the gums, buccal cavity and front part of the mouth for about 1-2 minutes. In case of a sore throat, gargling is recommended. If possible do not rinse with water.

Application

- Remove the dosing cup from the bottle head.
- Unscrew the bottle cap.
- Dosage in the cup to dispense 5-10 ml.
- Apply the solution to the oral cavity.
- Close the bottle after use.
- Date the bottle after opening.
- Store in a dry place, away from light, at room temperature (up to 12 months after opening).

C**RheoDol® Oral Hygiene Gel plus****Oral wound care**

After application of the mouth rinse (Liquid), apply pea-sized portion of gel to the oral cavity, tongue and mucous membranes. Apply several times a day in case of xerostomia (dry mouth) and inflammations such as mouth ulcer for moistening and anti-inflammatory measure. In case of a heavily coated tongue for tongue cleaning using cotton or oral swabs to carefully loosen and remove coatings.

Application

- Remove lid from the bottle.
- Remove safety nipple.
- Apply pea-sized portion of gel by squeezing the bottle head on swap.
- Apply the gel to the oral cavity.
- Write the date of opening on the bottle.
- Store dry, away from light at room temperature (12 months after opening)



ELISCHA Medical GmbH

Sonnenallee 17-21

DE 06766 Bitterfeld-Wolfen



ALPRO MEDICAL GmbH

DE 78112 St. Georgen



ACTO GmbH

DE 31613 Wietzen

RheoDol®.com
Medizinpflegeprodukte ELISCHA

